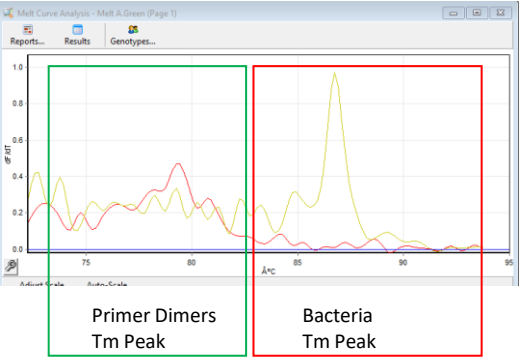


LINA + qPCR – Bacteria, Resistance, Fungi

	Positive blood culture 2µl BAL 20µl			LINA		Invert several times or vortex																
	IPC 20µl (Optional) Spin down the tube before use!			LINA		Invert several times or vortex																
	From the LINA – solution 20µl per reaction		“Ready-to-use” PCR- master mix 	Colours: Red – Bacteria (16S) Green – Fungi (28S) Yellow – Resistance Blue – IPC		Close PCR-master mixes																
Load PCR - device 			PCR - protocol: 95°C 2 min. 45 Cycles: 95°C 10 sec. 56°C 10 sec. 72°C 30 sec. 75°C 1 min. Melt (only qPCR): 75°C to 95°C, increment 0,3°C steps 10 sec. 25°C (hold)				Start PCR + plate read Fluor: SYBR Green															
PCR interpretation (only qPCR) 	The conditions in the table below must be met for a PCR result to be identified as positive. See the IFU, Pathogens xB for more information on PCR interpretation.					<table><tr><th></th><th>Ct-Value</th><th>Melting Temperature</th></tr><tr><td>Bacteria</td><td>< 34</td><td>80°C to 90°C</td></tr><tr><td>Fungi</td><td>< 40</td><td>80°C to 94°C</td></tr><tr><td>Resistance</td><td>< 34</td><td>80°C to 90°C</td></tr><tr><td>IPC</td><td>< 34</td><td>86°C ± 1°C</td></tr></table>		Ct-Value	Melting Temperature	Bacteria	< 34	80°C to 90°C	Fungi	< 40	80°C to 94°C	Resistance	< 34	80°C to 90°C	IPC	< 34	86°C ± 1°C	
	Ct-Value	Melting Temperature																				
Bacteria	< 34	80°C to 90°C																				
Fungi	< 40	80°C to 94°C																				
Resistance	< 34	80°C to 90°C																				
IPC	< 34	86°C ± 1°C																				


Storage: -25° to -18°C

hybcell – Bacteria, Fungi or Pathogens

Unpack hybcell



Insert into
rack: “click”

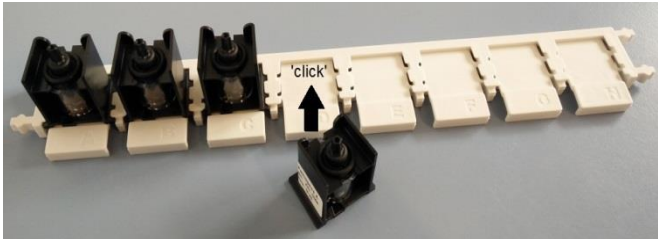

	● Bac. PCR	● Res. PCR	● Fun. PCR	● IPC PCR
hybcell Bacteria	•	•		•
hybcell Fungi			•	•
hybcell Pathogens	•	•	•	•

Combine up to **maximum 3** of the desired amplicons (30µl each) into any one of the amplicon tubes.
Pipette 30µl of the PPE-Additive into the mixing tube.

PPE-Additive



PCR amplicon:
Fungi



Pipette up and
down to mix

PCR amplicon:
Bacteria
40µl



100µl mixture of PCR
amplicons and PPE-Additive

The final volume of the mixture is dependent on the PCR-Boxes
required for the test!

Different examples of amplicon combinations suitable for the loading of the hybcell

Results of PCR				Amplicons transferred into hybcell
Bacteria	Fungi	Resistance	IPC	
pos.	pos.	pos.	pos.	Bacteria + Fungi + Resistance
pos.	pos.	neg.	pos.	Bacteria + Fungi
pos.	neg.	pos.	pos.	Bacteria + Resistance
neg.	pos.	neg.	pos.	Fungi + IPC

hybcell – Bacteria, Fungi or Pathogens

Fill the hybcell with the
(positive) PCR-amplicon and PPE-Additive mixture

Pipette the entire volume
gently and at once!





Insert the tip of the
pipette deeply into the
hybcell!

Try not to wet the
hybcell's inside
margins!

Once finished, close the
hybcell with the lid.

Create a run and start

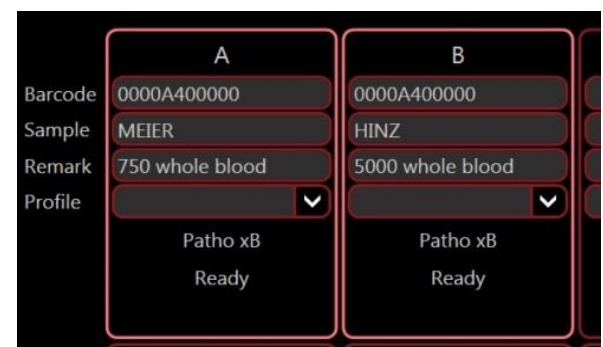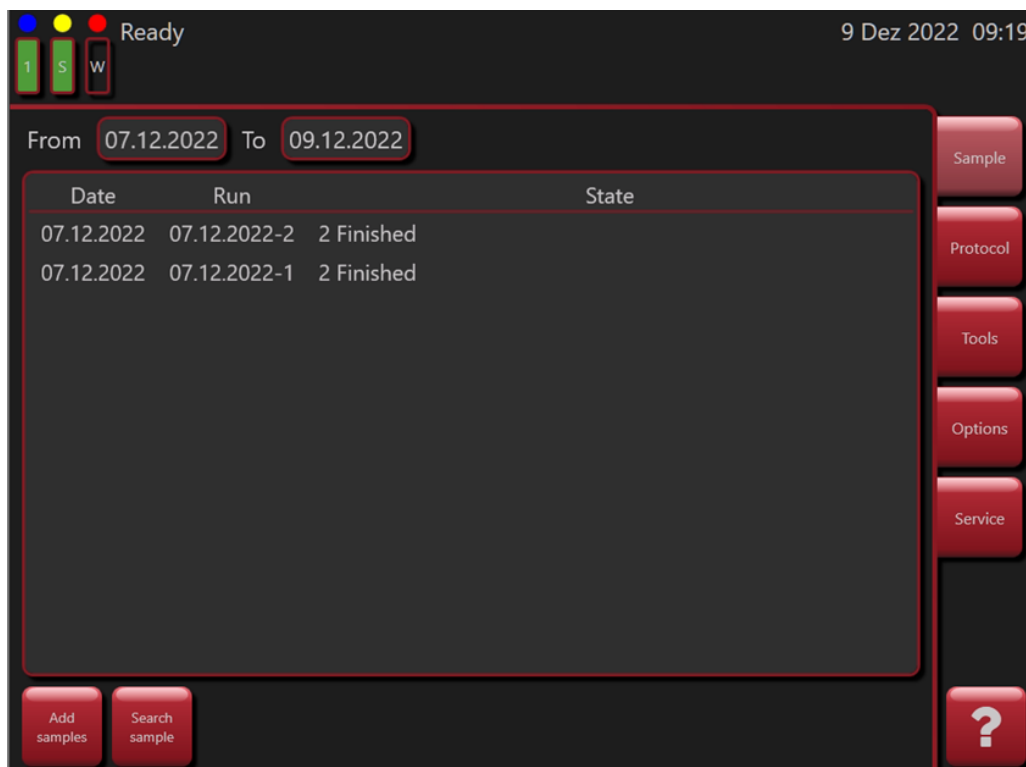

1. Create a new sample ("Sample" screen)
2. Insert data (sample ID, hybcell ID)
3. Select samples and start run ("Sample" screen)
4. Insert the rack (Barcode facing inwards) and confirm

